OtoVita[®] dry uv

Electronic drying station for hearing aid systems.

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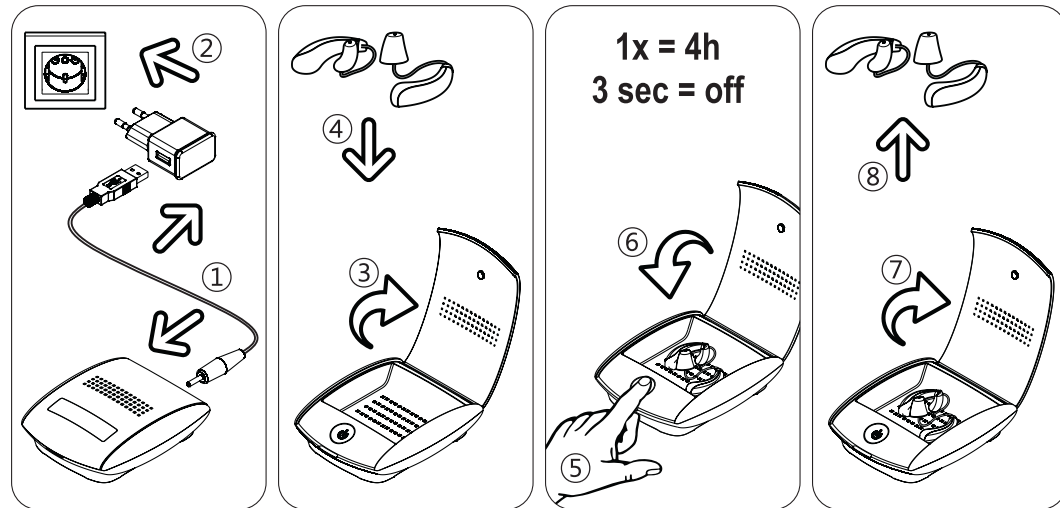
Dries and cares your hearing systems

Moisture and ear wax can impair the function of hearing systems and cause failures. The OtoVita® dry uv offers optimal care. Comfort drying at 50 °C extends life time significantly. Ultraviolet light (253,7nm UV-C) eliminates 99,9 % of germs and bacteria. This clinical cleanliness prevents ear infections risks and protects both your hearing systems as well as your health.

Using first time

1. Insert the USB-cable into the 5 V power supply and the round plug connector into the jack on the rear side of your drying station
2. Then plug the power supply into an AC power socket ⓘ The OtoVita® dry uv is ready to use

Operation

3. Open the lid of your OtoVita® dry uv
4. Place the with wet wipes pre-cleaned hearing systems with the battery-door open (if possible) into your drying station
5. Briefly touch the sensor button and the comfort drying program (drying + UV-C) is starting automatically
 - The 4 hours drying program with approx. 50 °C warm air is fully processor controlled ⓘ LED permanent light
6. Close the lid
 - Ultraviolet light (253,7 nm) eliminates germs and bacteria to 99.9% within 4 minutes ⓘ LED blink light
 - After 4 min the UV-C light switch off automatically, the drying process continues ⓘ LED permanent light
 - Within the last 4 minutes of the 4 hours drying program, a second UV-C hygiene runs ⓘ LED blink light
 - After 4 hours the drying station automatically shutdown to stand-by ⓘ LED switch off
7. Open the lid again
8. Remove the dried and hygienically cleaned hearing systems

Maintenance and operation

- We recommend wet wipes for cleaning your OtoVita® dry uv
- You can interrupt the drying process by holding the sensor button for more than 3 seconds
- The OtoVita® dry uv is very energy efficient; unplug the power supply only for prolonged absence

Warnings

- Avoid contact with water
- Don't use the OtoVita® dry uv near basins etc.
- In the event of a malfunction, unplug the power supply and take the drying station to your retailer
- Don't disassemble the drying station and use only the supplied 5 V power adapter
- Children must be supervised to ensure that they don't use the drying station as a toy
- The drying station is not to be used by persons who have limited physical, sensory or mental abilities or who do not have sufficient experience and/or knowledge of the drying station, unless they are supervised for their own safety by a responsible person, or receive instructions on how to use the drying station
- CAUTION: Do not look into the UV-C light, as this can harm your eyes and your skin
- The drying station has an automatic room and temperature modulation as well as safety shutdowns, when the lid is opened during UV-C program
-  Do not dispose of electrical appliances as unsorted municipal waste; use separate collection facilities
- Contact your local government for information regarding the collection systems available
- If electrical appliances are disposed of in landfills or dumps, hazardous substances can leak into the groundwater and get into the food chain, damaging your health and well-being

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